SUOMI

Membrasin Vaginal Vitality Cream

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ENGLISH

Membrasin Vaginal Vitality Cream

Membrasin Vaginal Vitality Cream is intended for treating dryness and maintaining the health of the mucous membranes of the vagina and the external female genitalia. The sea buckthorn oil in the cream treats and protects the mucous membranes and the sensitive skin of the intimate areas, and the hyaluronate moisturizes. The cream nourishes the mucous membranes and eases the symptoms related to dryness of the vagina and the external female genitalia, such as the sensation of dryness, irritation and sensitivity, as well as pain experienced during exercise. The pH value of the cream corresponds to the pH value of the mucous membranes of a healthy vagina. Membrasin Vaginal Vitality Cream is a hormone and perfume free medical device manufactured in Finland.

Instructions for use

The cream is applied to the vagina and the external genitalia once a day, as well as to the external genitalia after washing. The cream is inserted into the vagina using the applicator included in the package and applied to the external genitalia using clean hands. For optimal convenience,

we recommend applying the cream in the evening, which will enable the cream to stay in place better during the night and minimize any discomfort due to the cream running.

Application to the vagina (in the evening): See the picture

• Use the applicator provided in the package

1. Screw the applicator onto the nozzle of the tube, squeeze the cream until it reaches the mark (approx. 2.4 g, indicated by the black line on the side of the applicator) and unscrew the applicator from the tube

2. Insert the applicator into the vagina and apply the cream

• You may also use clean fingers to apply the cream. During pregnancy, apply the cream without using the applicator.

3. • Wash the applicator carefully with warm water after use. The easiest way to clean the applicator is described below: Pull the plunger out of the applicator tube and wash the applicator tube and plunger separately. Dry using toilet paper or paper towels and leave to dry on a clean surface.

Application to the external genitalia (in the evening and after washing): Wash your hands carefully and squeeze a small amount of cream out of the tube, a blob the size of your fingertip (approx. 0.2–0.4 g). Apply to the mucous membranes and skin of the genitalia

Warnings and directions

• If the mucous membranes are very dry, using the cream the first few times may result in temporary irritation or stinging • Do not use the cream if you know you are allergic/sensitive to any of the ingredients • If you experience symptoms of allergy/sensitivity, such as delayed and/or long-term sensations of itching or burning, stop using the product. You must also stop using the product if you experience any other adverse effects. If necessary, contact pharmacy staff or a healthcare professional • If you have an infection/inflammation in the affected area, do not use the cream without consulting a healthcare professional • Only use the applicator provided for applying the cream • Always wash your hands before using the product • Do not use the product directly after childbirth • Do not use the applicator during pregnancy • Stop using the product if your symptoms are not alleviated. The cream usually starts to alleviate symptoms after approximately two weeks of regular use • Do not use the product during protected sex. The oils in the cream may affect the durability of the condom. • Do not use the product after the expiration date marked on the tube. After opening, the product can be used for 6 months (☞ 6 months), but only up to the expiration date • Do not use the cream if its texture has altered significantly or if the tube is not intact • Do not use a damaged applicator • Keep out of reach and sight of children • Keep at room temperature

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