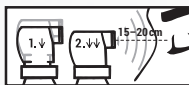
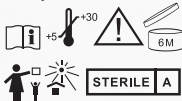
MEMBRASIN®
VISION VITALITY SPRAY

With Sea Buckthorn Oil

Dansk Nederlands
Deutsch Norsk
English Suomi
Français Svenska

AROMTECH LTD.
VETURILLIORTINIE 1
95410 TORNIIO,
FINLAND
AROMTECH.COM

Last revision: 01/2019

0038E-15

CE 0537

membrasin.fi

ENGLISH: Membrasin Vision Vitality Spray® relieves dry eye symptoms and moisturizes the skin around the eyes. Always apply the spray on closed eyelids only. The microemulsion contains sea buckthorn oil that slowly spreads on the tear film. The oil strengthens the tear lipid layer to protect the eyes from drying. Hyaluronate supports the water binding capacity of the tear fluid and lubricates the eyes; it also prevents cell damage. Thanks to its special design the bottle can be used in any position. Developed and manufactured in Finland, Membrasin Vision Vitality Spray® is a registered Medical Device product with no alcohol, no preservatives, and no ingredients of animal origin. Suitable for sensitive eyes and daily use.

Instructions for use: Take off the protection cap. Before the first application, test the spraying mechanism a few times on your arm or in the air. Notice that there is a small step in the nozzle; to apply the spray, press the nozzle fully down. Hold the bottle approximately 15–20 cm (6–8 inches) from the eyes. Close both eyes and then apply 1–2 sprays on the eyelids. Put the cap on. You may use the spray 1–4 times a day or more frequently, if needed and while wearing contact lenses. If you are using other personal care products or make-up in the eye area, let them settle for at least 10 minutes before applying the spray 20 cm (8 inches) from the eyes. Or, use the other product not earlier than 10 minutes after applying the spray. Use the contents within six (6) months from opening and by the expiry date indicated in the stamp on the bottle. (≅ 6 months)

Precautions and warnings. No specific contraindications. Do not use if you are sensitive to any of the ingredients. • Do not use if the bottle is damaged. The bottle contains a sterile bag that shrinks due

to usage. • To avoid contamination, do not touch the eye or skin with the tip of the nozzle. • Do not touch the tip of the nozzle with fingers. • If the spray gets directly in the eye, it may cause temporary uncomfortable feelings. If needed, rinse the eyes with clean water. • Leave at least 10–15 minutes between applications of the spray and an ophthalmic medication. • Keep out of reach and sight of children. • Store at 5–30 °C, protect from freezing, direct sunshine and heat. Do not use the product in case it is frozen or has been stored in hot conditions for a long time. • Discard the empty bottle appropriately. Due to intentional overflow, a small volume of the solution will remain in the bottle.

Composition: Sea Buckthorn Oil 0,4 %, Sodium Hyaluronate 0,02 %, Purified Water, Polyoxethylene Sorbitan Monooleate, Glycerol, Sorbitan Monooleate, Citric Acid, Monohydrate, Trometamol, Potassium Chloride, Sodium Chloride

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